

# EDAN 1201 Quick Guide

## STEP 1 Wake/Turn ON

- Press any key to wake device or hold down the POWER  button if the ECG machine is off.
- CHECK the RP (black or red box) has a steady red light, indicating it also has power.

## STEP 2 Skin prep and lead placement

- Prep skin and place leads according to your clinic guidelines.

## STEP 3 Enter Patient Info

- Press **F1** to open enrolment form
- Enter **Medical Record Number (MRN) aka Patient UR number** and use **Tab** to move to next field to enter all other patient details.
- Tab to **OK** and press **Enter** when finished.

### Additional codes

Symbol	Meaning	Placement
#	Urgent	At end of Patient ID

## STEP 4 Final check & run test

- Check the **Patient info** and **Time** are correct
- Check leads are connected and trace quality is good
- Press **PRINT/STOP** button to run ECG test
- When done, ECG will print and also transmit to CardioScan for reporting.



# Troubleshooting

## Check connection

- If you see this error message, ensure the white modem and black/red RP devices are plugged in.
- Then press **Enter** to return to the trace screen.



## Clear File manager

- Press **F6** to open new screen
- Press **F1** to open **File Manager**
- Press **F1** to select **TransAll**
- You will see a processing prompt
- Once **File Manager** is clear, press **ESC** to return to the trace screen
- The EDAN is now ready for use again.



## Persistent flickering green light

- If you notice the **green light** on the RP (black or red box) device continues to flicker for an extended time after completing an ECG (10+ mins), there is likely a transmission issue.
- Try moving the device to another area in the hospital to pick up a better Wi Fi signal.



### Online EDAN help and support

Scan the QR code with your iPhone camera and click the link, or use your Android QR code reader.

[cardioscan.co/EDAN-help](http://cardioscan.co/EDAN-help)

Helpdesk 1300-227-346

